
Nano-Check™ Influenza+COVID-19 Dual Test

- ☑ FDA 510(k) Cleared (k243561)
- ▼ FDA CLIA Waived (CW240029)
- ✓ Anterior Nasal Swab Specimen
- number of positive samples
- ☑ Results in 15 minutes
- ☑ No Extra Steps
- Made in USA





Easy/Simple To Use

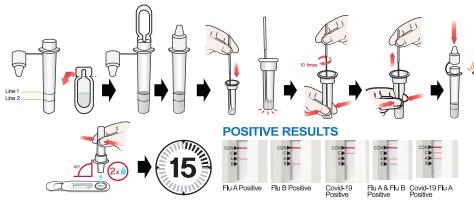

Contents

- Test Devices
- Ampules and Empty Reagent Tubes
- Sample collection swabs
- Positive control swab
- Negative control swab
- Instructions for Use
- Quick Reference Instruction

Performance Characteristics

SARS- CoV-2	Comparator RT-PCR		T-4-1
	Positive	Negative	Total
Positive	232	3	235
Negative	33	1,701	1,734
Total	265	1,704	1,969

Positive Percent Agreement = 87.6%	
Negative Percent Agreement = 99.8%	'n

1	Flu A	Comparator RT-PCR		T-4-1
		Positive	Negative	Total
1	Positive	417	6	423
	Negative	63	1,483	1,546
1	Total	480	1,489	1,969

Positive Percent Agreement = 86.9% Negative Percent Agreement = 99.6%

	Flu B	Comparator RT-PCR		
			Negative	Total
	Positive	99	5	104
	Negative	15	1,850	1,865
	Total	114	1,855	1,969

Positive Percent Agreement = 86.8% Negative Percent Agreement = 99.7%

Refer to IFU for additional LOD/Analytical Reactivity/Clinical Performance Information

Ordering Information Cat. No.: MD8154 (25 Tests)



AqueousDx.com



For a quote and orders: orders@aqueousdx.com