



# THE ULTIMATE CRYOSURGERY DEVICE


CryoProbe sets a new standard for the cryogenic treatment of benign skin lesions, offering the ultimate in control, convenience, and cost-effectiveness – as well as an outstanding patient experience.

This high-performance, ergonomic cryotherapy pen dispenses a fine jet of nitrous oxide under high pressure, allowing you to work with millimeter precision.

The CryoProbe XP is our most performant model, and is suitable for intensive and varied use.


## Key features

- Ergonomic design for superior control
- Leading-edge microtechnology for controlled gas flow
- Takes easy-to-load 8g or 16g N<sub>2</sub>O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 4 applicators
- Suitable for intensive and varied use
- Certified **CE1639** **FDA** **MPSAP**

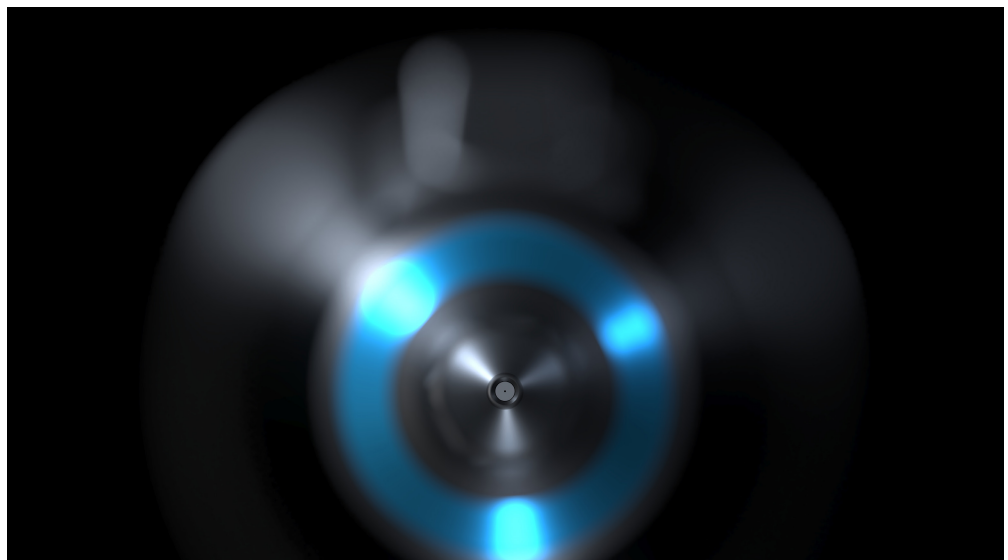





# APPLICATORS

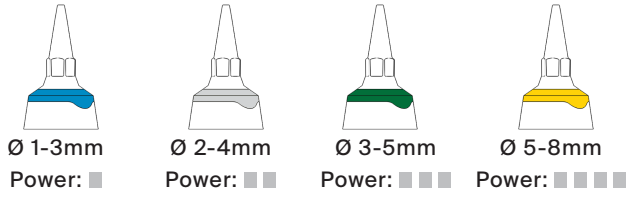

### Continuous flow time with a 16g cartridge

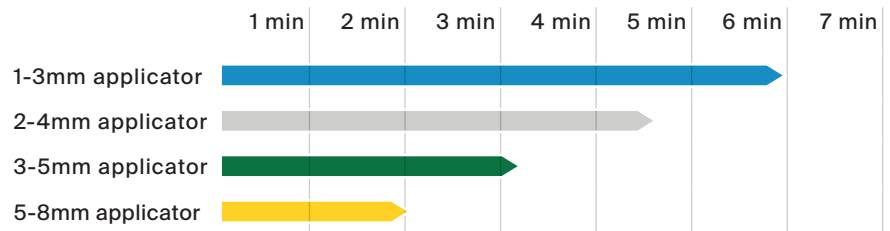

### Continuous flow time with an 8g cartridge

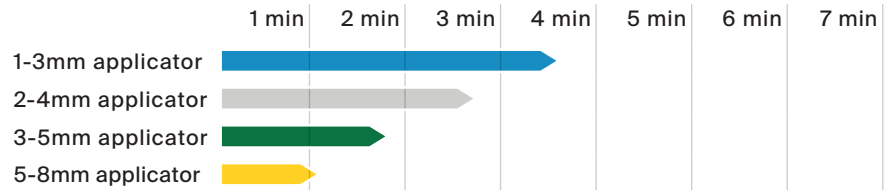

# PRODUCT INFORMATION

### Tutorial

Scan or click:



### Device

The CryoProbe XP is suitable for all treatment situations, whether more delicate or stronger freezing power is required. The device can be used with 8g or 16g cartridges, and with the complete range of applicators, making it a highly versatile instrument.

### Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

### What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, granuloma annulare, molluscum contagiosum and seborrheic keratosis.



## CONTENTS OF THE STARTER SET

### Device

CryoProbe XP

### Applicators

- 1-3mm applicator
- 2-4mm applicator
- 3-5mm applicator
- 5-8mm applicator

### Cartridges

6x 16g cartridges

### Accessories

- Metal carrying case
- Forceps
- Converter for 8g cartridge
- Device stand

### Documentation

- Treatment overview
- User manual

## BASICS

### Dimensions

- Device: 22.6 x 3.3 x 3.3 cm
- Carrying case: 28.8 x 22.8 x 6.7 cm
- Starter set: 38 x 30 x 7.5 cm

### HS codes

- 90189084 (device)
- 28112930 (cartridges)

### Weight

- Device with a full cartridge: 300g
- Starter set: 2,470g

